6007

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Mode d'emploi

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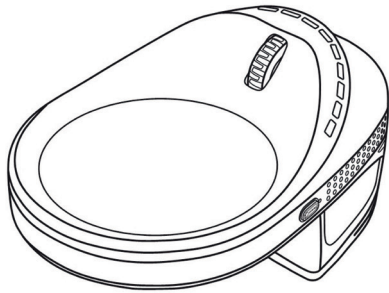
Instructions for use

Technical specifications

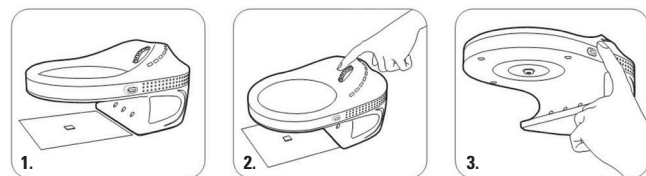
Display: Super TFT
Housing: Reinforced ABS
Dimensions: 13 x 87 x 64 mm
Screen diameter: 2.8 inches (71 mm)
Battery: 3.5 hours continuous use / 2.5 hours charging time
Light source: 7 white LEDs / 4 UV LEDs
Input: USB-C / DC5V <2A
Weight: ~ 30 g

Précautions

1. Use only at temperatures between -15 and 55 °C.
2. Avoid contact with liquids.
3. Do not place the device near heat sources, open flames, or flammable or explosive gases or liquids.
4. Prevent dust from entering the screen and the interior of the product.
5. Avoid impacts and significant vibrations.
6. Do not charge for more than 12 hours to avoid reducing the battery life. Note: During charging, it is normal for the outer casing of the microscope to become slightly warm.

Cleaning: It is necessary to clean the screen and the casing regularly to ensure the device remains clean and functions correctly. If the screen is dirty, wipe it gently with a soft cloth.

2

Please place the magnifying glass on a stable surface, press the power button on the side to switch it on, then place the object you wish to view directly under the lens.

* Press the ON/OFF button for approximately 3 seconds to switch the device on or off. This device has a fixed focus and does not require repeated focusing, making it easy to use.

You can select the magnification level from 4 options by turning the dial. (1x-2x-3x-4x)*

Turn up = Zoom in, turn down = Zoom out
Press the dial for photos, hold it down for videos.

* corresponds to:
8x-11x-13x-16x

You can switch between lighting modes by pressing the ON/OFF button once:
- white light
- UV light
- side white light
- no light

The light intensity can be adjusted by pressing and holding the ON/OFF button in any lighting mode. To switch off the LED magnifier, switch to the no-light mode, then press and hold the ON/OFF button.

4

Please read these instructions carefully before using the system!

Notes

Please contact us if any accessories are missing.
Design and technical specifications are subject to change without notice.
The product names, service names, etc. described in this manual are registered trademarks of the relevant companies.

Warning

- Keep out of reach of children.
- Do not use or store this product or its accessories in hot environments such as cars, heaters or cooking appliances, or in direct sunlight.
- Do not use in dusty areas.
- Do not use if damaged.
- Do not use thinners or other organic solvents to clean the product.
- Do not store the microscope in a place where it may fall.

Note

The illustrations used in this manual are for reference only. Please refer to the actual product.

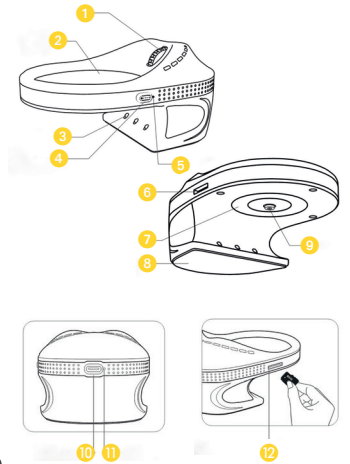
Dispose of the appliance in accordance with all local, regional, national and international regulations.
It must not be disposed of with household waste.

Bergeon SA - Allée des Défricheurs 4a - 2300 La Chaux-de-Fonds - Switzerland
Tel. +41 32 933 60 00 - info@bergeon.ch - www.bergeon.ch v 1.0 / 2026-05

1

Product description

- 1 Scroll wheel
- 2 2.8-inch screen
- 3 Parking light
- 4 ON/OFF button (light adjustment)
- 5 Reset
- 6 SD card slot
- 7 Light ring
- 8 Anti-slip spacer
- 9 Camera
- 10 USB-C port for charging and connecting to a large screen
- 11 Red light - Charging in progress
Green light - Charging complete
- 12 SD card slot, supported memory cards (max. 64 GB)

Fully compatible with multiple devices, supports chargers, power banks and computers for recharging visual aids.

The device switches off automatically after 10 minutes of inactivity.

If the device freezes, use a SIM card eject tool to restart it. The reset button is located to the right of the power button.

3

Usage

1. Use the supplied charging cable. Plug end C into the device's charging port and end A into the computer.

2. When 'PC Camera' appears on the screen, the magnifier cannot be operated. You must control it from the computer.

3. The magnifier functions as an external USB camera.

Note: Before using the magnifier, please insert the SD card to ensure photos and video files are saved. (Maximum supported memory card: 64 GB)

When the screen with our logo appears, press and hold the power button again to enter date setting mode, then click the toggle button to change the date.

Photo Mode

Click the scroll wheel to switch to image lock mode. The image is frozen. Click the scroll wheel again to unlock the screen and automatically save the image to the SD card.

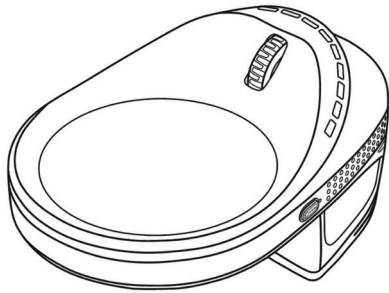
Video Mode

Press and hold the scroll wheel to switch to video mode. A red dot will flash on the screen to indicate that the video is being recorded and saved to the SD card.

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