

# Ear piercing training manual



with the Caflon Blu Disposable  
Cassette Ear Piercing System



*the safe way to pierce ears*



## **Manufacturer and Distributor of Ear Piercing Equipment**

**This manual will cover the following aspects of ear piercing**

**Information about Caflon**

**Caflon earrings**

**The anatomy of the ear: which area of the ear to pierce**

**Contra indications of ear piercing**

**Step by step ear piercing using the Caflon Blu System**

**Aftercare**

**Potential problems and further information**

---

# Introduction

Welcome to the Caflon Blu ear piercing training manual. The aim of this manual is to provide the user with clear and precise information on how to pierce a client's ears using the Caflon Blu Ear Piercing System.

Ear piercing using Caflon Systems is a safe and useful profit making service requiring little in the way of financial investment, using minimal amounts of space, and taking just a short amount of time. Profit margins are also good making the Caflon Ear Piercing Systems a wise and safe investment.

read on...





**CAFLON Ltd.**



A British based company involved in the manufacture and distribution of high quality ear piercing equipment and hypo allergenic fashion earrings.



One of the largest manufacturers and distributors of ear piercing equipment in the world with distribution in over 100 countries.



The Caflon collection of ear studs includes all stones and shapes of fashion. All are precision turned on Swiss Escomatic Machines.



Caflon has available a wide range of sales and marketing aids, Ear Care Solution and antiseptic wipes, as well as starter kits designed especially for the ear piercing beginner.

CAFLON

intern

accept

as

of the

world



## Range of earrings


- \* Caflon ear studs are made from hypo-allergenic material meaning they do not cause allergies or irritations.
- \* And are available in many different styles and sizes such as gold, 24ct gold plate, silver plate and Titanium.
- \* All studs are supplied in validated pre-sterilised blister packs using 'Ethylene Oxide' Gas and Gas permeating paper.
- \* Every pair of earrings, therefore, has been individually sterilised.
- \* Compliant with EC Nickel Directive 94/27/EC

# Outer ear

## What area of the ear can you pierce with Caflon Blu

- ✿ The lobe is the soft area of the ear and consists of skin covering connective fatty tissue. It is the area of the ear which can be pierced. The healing period is 4-6 weeks.
- ✿ The cartilage area is a type of elastic tissue covered by skin and fine hair. The area which can be pierced is the flat upper part of the cartilage area indicated by the diagram below. Due to the nature of piercing the cartilage it is very important to advise the client that though piercing this part of the ear can be safely done with the Caflon Blu system, extra care must be taken in caring for this type of piercing. The healing period is at least 12 weeks.
- ✿ The Caflon Blu Ear Piercing System is not specifically designed to pierce any other part of the body and should only be used to pierce the shaded areas of the ear indicated in this diagram.

### Anatomy of the outer ear ►

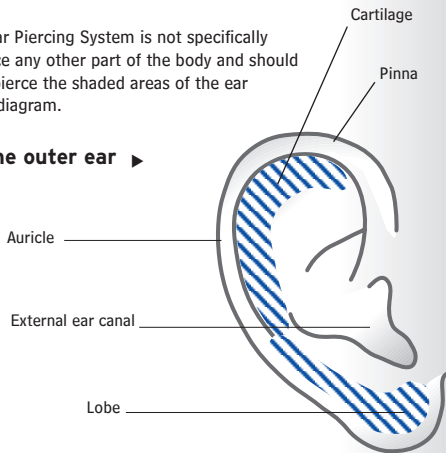

## Contra indications of ear piercing

- ✱ Contra indications for ear piercing normally involve the client having a condition which means they might be prone to infection or slow to heal from any potential infections, a physical injury, or some kind of obstruction.

**If a client is suffering from any of the following then they should seek medical advice before you pierce their ears.**

Circulation disorders

High or low Blood Pressure

History of Thrombosis / Embolism

Epilepsy (Epileptics must be accompanied by someone if you are to pierce their ears)

Pregnancy

Diabetes

Dysfunctions of the Nervous System

Skin disorders

Bruises or recent Haemorrhage or Swelling

Scar tissue or Keloids

Warts or Moles (you may pierce through freckles)

Cuts, Abrasions or recent operations

**If in doubt do not pierce.**


# Equipment

In order to ear pierce using the Caflon Blu System you will need to be familiar with the equipment you are to use, namely the ear piercing instrument, the studs, the single use clasp retainer/ear guard and stud holder, and the blister pack itself.

The clasp retainer/ear guard and stud holder (see next page) do not come with the instrument. They are pre-sterilised, single use disposable items that are packaged together with the sterilised blister pack holding the stud and clasp.

**IMPORTANT!** Never use the same clasp retainer/ear guard and stud holder on more than one client.

## The Caflon Blu instrument ▼

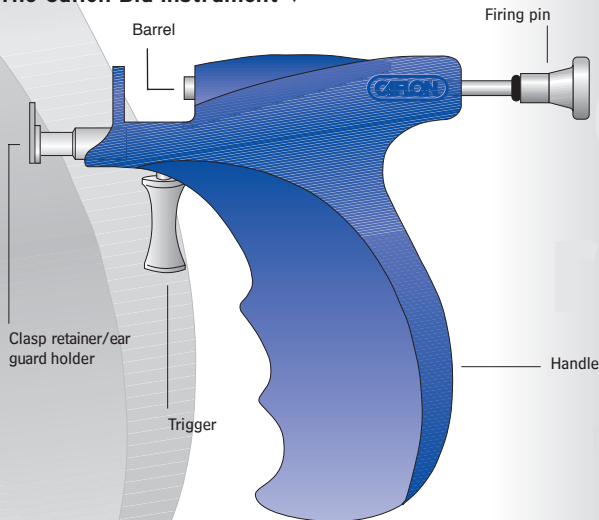




# Contents of Blister pack


Clasp ▼



Clasp retainer/  
ear guard ▼

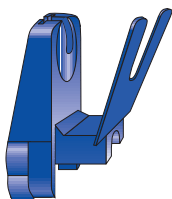


Stud ▼

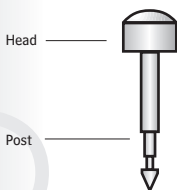


Stud Holder ▼


Mini size(Blue)      Regular size(White)

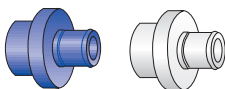

Stud Holder remover ▼

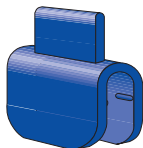

Blu plastic mount ▼

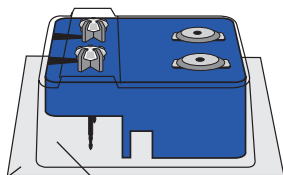

Tyvek paper  
Plastic blister pack  
(Clasp retainer/ear guard and stud retainer  
located inside blister pack.)

# Step-by-step ear piercing

## Step 1: preparing yourself and the client

- ☛ Clients will normally fit into 'nervous' or 'non nervous' categories. Obviously 'nervous' clients are potentially more difficult to deal with. However most of their anxiety will usually be associated with the thought that the piercing may be painful. They might have had a painful ear piercing experience previously.  
These kinds of clients need to be relaxed through reassurance that the Caflon Blu System allows for pain free ear piercing.
- ☛ Seat the client on a stool or chair which is at the right height for you to complete the piercing. You should not have to reach up or down in order to pierce the client's ears.
- ☛ Ask the client to select the style of stud to be used for piercing from the Caflon sample board
- ☛ Explain the piercing procedure and have the client read and sign the ear piercing application and after-care instructions.
- ☛ Check thoroughly for contra indications. If in doubt do not complete the piercing.
- ☛ Now wash your hands thoroughly or scrub them with a chlorahexidine based surgical hand cleaner, dry them and then ideally you should put on single use disposable gloves as an additional precaution.
- ☛ Have the client remove any earrings they are wearing.
- ☛ Clip back any hair away from the ears.
- ☛ Check the client's ears for epidermoid cycts which are embedded skin tissue enclosing foreign debris. When you pinch the ear, if you feel a small, hard lump that can be moved, it is an epidermoid cyst. If you can see and feel a lump that does not move, it is scar tissue. Cysts are not visible and can be moved, scar tissue is visible and does not move. Never pierce through a cyst as it will cause it to rupture releasing debris into the surrounding tissue which can cause infection. You should also not pierce straight through Keloids. which are large visible lumpy scar tissue. The rule to follow is:-  
**Keloids/scar tissue, pierce 3 - 4mm away from it.**  
**If hole is open, pierce 1cm away from it.**

You will need to pierce further away if the hole is open because an inaccurate piercing could lead to one large hole in the lobe. Additionally, it avoids the clasps overlapping at the back of the ear.

- ✿ You may pierce through an old hole as long as there is no heavy scar tissue present.
- ✿ After examining the ear(s) thoroughly cleanse the area to be pierced front and back with an antiseptic wipe, be sure to use a separate wipe for each ear. This will safeguard against infection.
- ✿ Allow 2 minutes for the solution to soak in and the ear to dry thoroughly.
- ✿ During this time you should demonstrate to the client the noise the instrument will make when you eventually pierce their ears. If familiar with this sound they are unlikely to move during the piercing, lessening the likelihood of an inaccurate piercing.

To demonstrate the noise:

- ✿ Take the instrument in your preferred hand.
- ✿ Pull back hard on the firing pin until it locks.
- ✿ Place the instrument close to the client's ear.
- ✿ Tell them that they are about to hear the noise the instrument will make.
- ✿ Fire the instrument by squeezing the trigger.
- ✿ Once the antiseptic lotion is dry you should use the non toxic marking pen to mark a spot on the ear where it will be pierced. No other type of pen should be used.
- ✿ Using a mirror, show the client the spot marked on the ear.
- ✿ Only if the client agrees the mark is in the correct place should you proceed with the piercing.
- ✿ If they state a preference for another spot then wipe off the original mark using another antiseptic wipe and repeat the previous marking instructions.
- ✿ Only when the client is happy with the positioning of the mark should the piercing take place.

# Preparing the equipment

- ✿ You will need to undo the blister pack on a clean and level surface.
- ✿ Tearing along the serrated edge of the blister pack, separate one pair of studs from the rest of the pack.
- ✿ On your clean, level surface turn the pack upside down so that the tyvek paper faces upwards and the plastic rests on the table's surface.
- ✿ Locate the corner in which the plastic has an arc, this is where you need to separate the tyvek paper from the plastic.
- ✿ Keeping the blister pack close to your level surface, peel the tyvek paper back slowly being careful not to spill the blue plastic clasp retainer/ear guard and stud holder onto the table or floor. If this does occur then the blister pack must be discarded and a new one used. This ensures sterility.
- ✿ Then cock the instrument by drawing the firing pin back until it locks into place. With gloves on both hands, take the small round blue (Mini)/white (Regular) stud holder from the blister pack and secure it onto the barrel of the instrument (see Fig. 1).
- ✿ Take the instrument in one hand and the blue clasp retainer/ear guard in the other. and firmly mount the blue clasp retainer/ear guard onto the ear guard holder of the instrument making sure that the flexible ear guard is facing upward and inward (see Fig. 1) pushed firmly down.
- ✿ Place the instrument onto a clean surface.



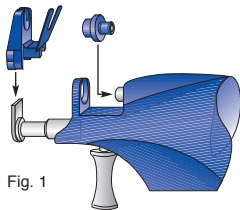


Fig. 1

- ✿ Next, hold the clear blister pack containing the plastic stud and clasp mount in one hand and carefully remove the blue mount ensuring not to touch the sterile studs and clasps.
- ✿ Holding the blue mount by the sides, the stud and clasps should be facing upwards.
- ✿ With the mount in one hand take the ear piercing instrument in the other and using the mount, insert the exposed head of the ear piercing stud into the round plastic stud holder previously inserted onto the barrel of the instrument then twist the mount upwards to release the stud from the mount leaving the stud in the plastic stud holder. (see Fig. 2).
- ✿ Ensuring that the instrument remains level, invert the plastic mount and slide the corresponding clasp onto the plastic clasp retainer/ear guard previously loaded onto the instrument (see Fig. 3).
- ✿ Place the plastic mount containing the remaining stud and clasp in the upright position on a nearby clean surface ready to be used on the second piercing if required.
- ✿ The instrument is now loaded and you are ready to pierce. Follow the instructions on piercing the client's ear on the next page.
- ✿ Once you have completed the first piercing, repeat the above process for the second piercing.

These two operations are extremely simple to perform but you may need to practice a few times to become proficient. Remember that once loaded, always keep the instrument in a horizontal position to prevent the stud and clasp from falling out.

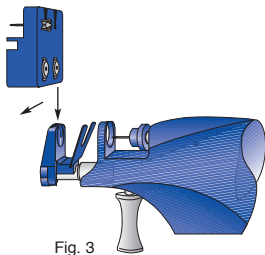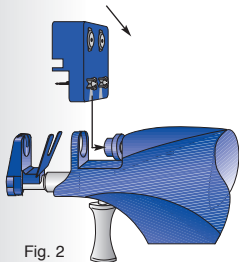

# Piercing the client's ear

## PIERCING THE LOBE AREA OF THE EAR

- ✿ Bring the loaded instrument up to the area of the client's ear to be pierced ensuring that the ear is placed into the saddle-like flexible ear guard (see Fig.5). If piercing the cartilage area see the instructions on piercing the cartilage area which follow.
- ✿ Make sure the instrument is straight and square to the ear otherwise the piercing will occur at an angle. It should be perpendicular and not tipped at an angle.
- ✿ Squeeze the trigger gently to halfway where you will find a biting point in the trigger mechanism. This will bring the tip of the stud closer to the ear. Check your alignment (make sure the point of the post lines up with your pen mark).
- ✿ When you are satisfied everything is correct advise the client to remain still. Squeeze the trigger the rest of the way back until the stud is fired into the ear.
- ✿ Hold the trigger in place and the instrument still for two seconds.
- ✿ Slowly release the trigger and carefully pull the instrument straight down. Do not pull it away from the ear (see Fig. 6).
- ✿ After reloading the instrument, repeat the above to perform the second piercing.
- ✿ Next, ask the client if they are OK to stand or if they need a few moments. If the client stands and feels dizzy, help them to sit down and have them wait 3 to 5 minutes before leaving.
- ✿ Advise on aftercare.
- ✿ After completing the piercing, wipe the instrument with an alcohol towelette or an approved cleaner.

## REMOVING THE DISPOSABLE STUD HOLDER AND CLASP RETAINER/EAR GUARD FROM THE INSTRUMENT



- ✿ To remove the clasp retainer/ear guard, simple slide it off the instrument and dispose it in the waste receptacle.
- ✿ To remove the round stud holder, place the studretainer clip over the barrel of the instrument (see Fig. 7). Next, pull the firing pin all the way back until the stud holder snaps loose. Release the firing pin and now remove the stud retainer clip and the stud holder. This action will cock the instrument so remember the disengage the firing pin by pulling the trigger making sure that fingers are free from the path of the barrel.

## PIERCING THE CARTILAGE AREA OF THE EAR

If you are piercing through the cartilage area of the ear, special care must be taken during both the piercing procedure and during the after care period.

- ✿ When piecing the cartilage area of the ear, only pierce the upper flat area. Never pierce through the curled edge of the cartilage area or too close to the edge of the ear as this may cause the cartilage to split. Also, never pierce through scar tissue in the cartilage area, avoiding all visible capillaries.
- ✿ Prepare the piercing area as instructed previous.
- ✿ Have the client tilt their head all the way back so that they are facing skyward. This action makes it easier to position the instrument for piercing while maintaining the instrument in the all important horizontal position which ensures that the clasp will not fall out before the piercing.
- ✿ If the client has very thick or heavily curled cartilage, you must pull the cartilage to "unroll it". This ensures that the instrument can be properly positioned.
- ✿ Once the instrument is positioned and ready for firing, follow the same procedure described when piercing the lobe previous.
- ✿ Repeat procedure for the other ear if required.

**IMPORTANT!** Alert your client that the healing time for the cartilage area can take up to 12 weeks. Reinforce the fact that due to the nature of piercing the cartilage area extra care should be taken to ensure proper after care is followed. If pain, redness or swelling persists, advise them to seek medical attention immediately. Failure to do so may lead to permanent damage to the ear.

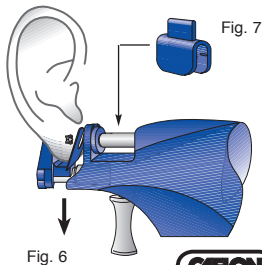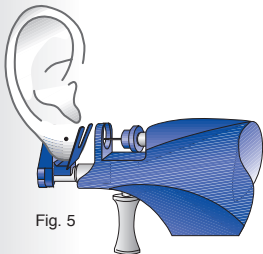

# Aftercare

If the instructions on how to pierce ears contained in this manual are followed stringently then potential problems following a piercing will have been minimised. However, the importance of aftercare cannot be overemphasised.

The client should be given the following advice for aftercare.

## To prevent infection

- ✿ Do not fiddle with the ears, avoid any unnecessary handling of the ears and studs.
- ✿ Do not replace studs with other earrings during the initial 6 week period (12 weeks for cartilage).
- ✿ Wash hands before touching the stud or ear and cleanse the pierced area twice a day without removing the earrings by lightly applying Caflon Ear Care Solution to the front and back of the pierced area.
- ✿ With clean hands, rotate the studs back and forth at least once a day.
- ✿ Do not allow soap or shampoo to build up behind the stud or clasp. Rinse well with the Caflon Ear Care Solution after bathing.
- ✿ Between cleanings, keep the pierced area DRY.
- ✿ Cover the ears when spraying perfume, antiperspirant, hairspray.
- ✿ Do not push the butterfly clasp along the post toward the ear. The Butterfly clasp should always be positioned at the end of the post.
- ✿ Minor pain/redness may occur immediately - this is normal. This should settle within 48 hours provided the client faithfully follows proper after care. If undue pain/swelling occurs advise the client to consult their physician immediately but warn them not to remove the studs until they have consulted their doctor.



### To prevent piercings from closing

- ✿ Do not take studs out for at least 4–6 weeks (12 weeks for cartilage).

### If infection or irritation occurs

- ✿ Should excessive redness, irritability, itching or soreness occur consult your doctor immediately. Do not remove the studs until you have consulted your doctor.

### General points

- ✿ After 4–6 weeks the stud can be removed from the ear lobe (12 weeks for cartilage) and another post type earring should be worn. The post should be 24 ct Gold Plated or Stainless Steel and this type should be worn for at least another 2 months following the healing period. No wire type earrings should be worn during this period. The wearing of heavy earring at any time may cause the piercing hole to stretch leaving a slit effect to the piercings.
- ✿ Caflon Ear Care Solution should be suggested to your client. This way you can be sure that the client receives the correct solution for aftercare.
- ✿ During the first year of piercing the client should avoid going longer than 24 hours without wearing earrings. This will ensure that the piercings will not close.
- ✿ If the client follows these aftercare instructions faithfully they will, in later years, have a small smooth round hole in each ear – never a slit.

## Potential problems

- ✿ Ear piercing is a relatively simple and straightforward procedure in which nothing can go wrong providing you stringently follow the instructions contained in this manual.
- ✿ The rate at which individual's heal after a piercing can vary enormously, but an average would be 4–6 weeks for lobe and 12 weeks for the cartilage.
- ✿ Perhaps the worst scenario you might expect is that the client will move as you fire the gun. This will inevitably result in an inaccurate piercing.
- ✿ If this happens do not remove the ear stud as the ear will bleed. Advise them to leave the stud in overnight, then remove the stud and let the hole heal over.
- ✿ This advice also applies to the client who decides, despite the fact that you've followed the instructions in the manual, that they do not like the piercing. Advise them to leave the stud in overnight, then remove the studs and allow the hole to heal over.

ear  
pier  
is  
a rel  
sim  
and  
stra  
proc

ring  
atively  
ple

ightforward  
edure



## Further information

Providing you have followed and understood the instructions contained in this manual, you should now be capable of piercing ears using the Caflon Blu Disposable Cassette System.

However, there are still a number of issues relevant to ear piercing which the piercing process itself does not cover.

## Ear piercing legislation

### Current legislation

- ✿ In England and Wales current ear piercing legislation falls under the Local Government (Miscellaneous Provisions) Act, 1982.

This means that the Local Authority has the power to implement the aspects of the Act that relate to ear piercing in their area.

- ✿ The Act provides that ear piercing may only be carried out by a person in premises registered by the Local Authority.
- ✿ You should always check with your local health authority to find out if there are any possible licenses, fees, regulations or requirements which need to be obtained or followed before performing any ear piercings in your establishment.

ear  
piercing  
may  
or  
be  
carried  
by  
premises  
registered  
by  
Local  
Authority

- ✿ The London area is different to the rest of England and Wales in that in the London area there are 2 other Acts under which boroughs may regulate skin piercing. The Greater London Council (General Powers) Act 1981 and the London Local Authorities Act 1991.

Anyone intending to pierce ears in premises in the London area should contact their Local Environmental Health Authority for advice.

- ✿ The Local Authority does have the power to inspect both registered premises and premises in which it suspects skin piercing is taking place without a registration.
- ✿ Fees for licensing and registration can vary enormously. Details can be obtained from your Local Authority.

### **The future of regulation**

- ✿ Legislation regarding ear piercing is currently under review. With the advent of much more sterile and hygienic ear piercing, such as the Caflon Disposable Cassette System, ear piercing is now much safer and carries no hygiene risk.
- ✿ In our commitment to constantly provide the best possible service to our clients, Caflon takes a pro-active approach to ear piercing regulations by contacting many of the individual Local Authorities throughout the United Kingdom to ensure that the best and the fairest possible regulations covering ear piercing are implemented by the different boroughs. Should you have any questions with regard to these matters, please contact us. Our professional and experienced staff will gladly assist you in this area.

## Insurance

Caflon equipment is covered by Product Liability Insurance. However, as an individual you will require your own insurance cover.

Many companies now require evidence of some form of ear piercing training such as that offered by Caflon before issuing insurance cover.

The Caflon training system is particularly relevant in this respect in that **models are used during training as opposed to ear cards**. This means that you will already be experienced when you pierce a client's ears for the first time. Should you wish to be included in our next training seminar in your area, please contact our Head Office to find out when the next Caflon training specialist will be in your area.

## Age restrictions

Do not pierce anyone under the age of 16 without the presence of a parent or legal guardian. Failing such may expose you to criminal charges of grievous bodily harm to a child despite the fact that the child consented without the permission of a parent or legal guardian.

A general rule to follow would be to exercise caution when piercing the ears of anyone under the age of 16. Such clients should be accompanied by an adult. Do not accept consent by telephone or letter. It is strongly advisable to have the parent of legal guardian read and sign the release form in your presence.

Though piercing the ears of small children is perfectly safe using the Caflon systems, one should always strongly caution the parent that due to the child's age, after care may be difficult for the child to follow. Parents should assume the responsibility of ensuring that the child's piercings are properly maintained and to seek medical attention at the slightest hint of a problem.

tion

piercing

s of anyone

er  
the age of **16**

**Caflon Limited**

[REDACTED]  
[REDACTED]  
[REDACTED]  
[REDACTED]  
[REDACTED]  
[REDACTED]  
[REDACTED]

---