

Assistance

The ultra-sound does not work.

Please check if the ultra-sound is installed as follows:



Key	Display indication	Remark
	<i>Service, WELCOME !!!</i>	Welcome to the service Program.
	<i>US uninst, 1, Toggle</i> Or <i>US inst, 1 , Toggle</i>	Ultra-sound is not installed.
	<i>US inst, 1 , Toggle</i>	Press button 1 Indication is changing between US inst and US uninst

Press button 1 till display indication shows you (US inst, 1, Toggle).

Press button



Check and reinstall programs 1 to 6 as required.