TISSIDental

PRECAST 1P MINI MED MAXI

Manuale di Istruzioni
Instruction Manual
Gebrauchs Anweisungen
Mode d'emploi
Manual de manejo

CAUTION

INSTALL THIS DEVICE IN A PROPER LOCATION
TO BE USE BY AUTHORIZED AND INSTRUCTED PERSONNEL ONLY

INDICE / INDEX

Declaration of conformity	3
Exploded draw	4
Spare parts list	5
Electrical diagram	6
Control panel	6

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ENGLISH

1	Scope of manual	16
2	General characteristics/expected use	16
3	Installation	17
4	Programming of cycle	17
5	Programming of start up	19
6	Baking cycle	20
7	Inspections	20
8	Power failures	20
9	Alarms	21
10	Environmental technical data	21
11	Ordinary maintenance	21
12	Warnings	22
13	Important information	22
14	Guide to common questions	22
15	Specifications	22
16	Notes	22

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TISSI reserves the right to change the specifications of this equipment without notice.

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Herewith we declare that the product is in compliance with the relevant requirements in the following directives:
 2006/42/EC (Machinery safety)
 2006/95/EC (Low voltage equipment)
 2004/108/EC (Electromagnetic compatibility)

Harmonized specifications applied:
 EN ISO 12100-1:2010, EN 61010-1:2010,
 EN 61326-1:2006

Authorised to compile the technical documentation:
 Tissi G. I.

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PRECAST – PRE-HEATING FURNACES

ATTENTION: do not place too many cylinders into the chamber – follow the indications specified below – the overload of the chamber cause malfunctioning and breaks of the heating element

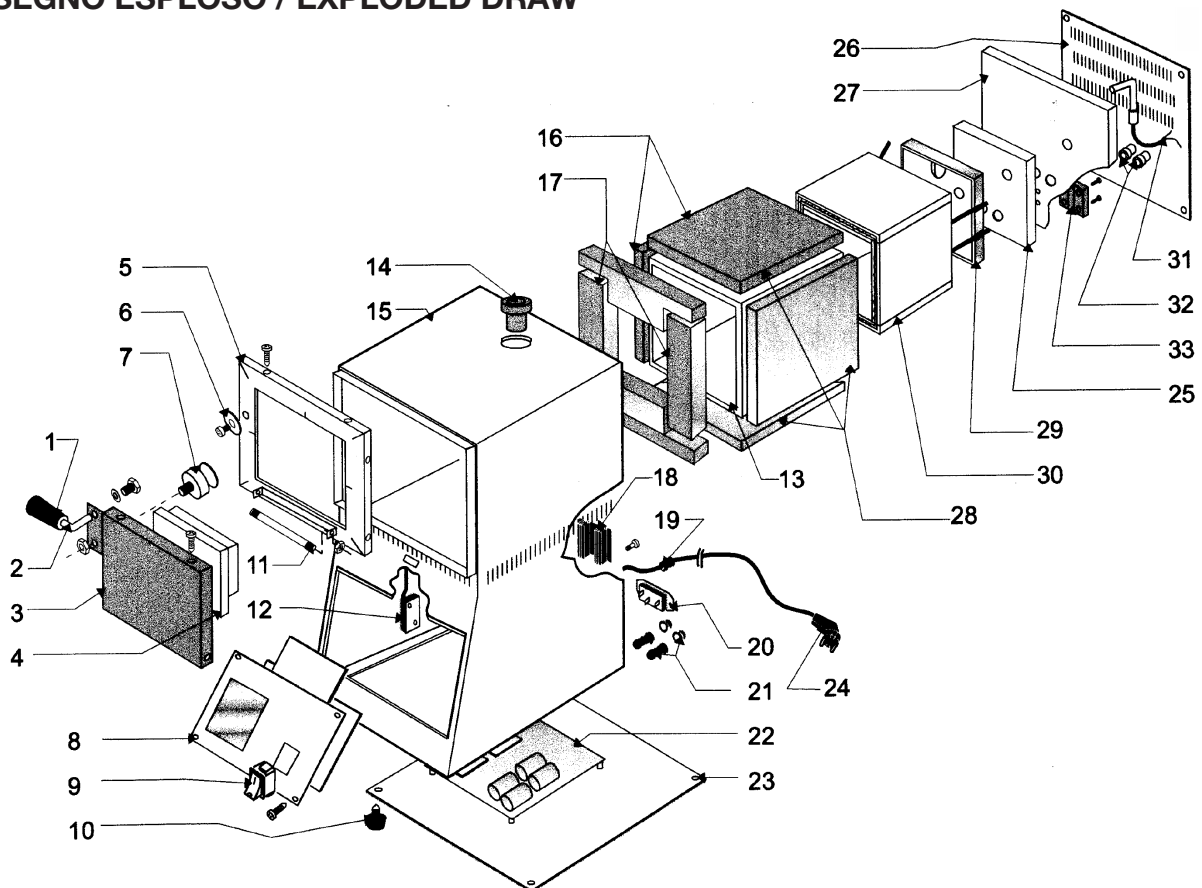
Precast mini: 6 cylinders 3x or 3 cylinders 6x
 Precast med: 9 cylinders 3x or 5 cylinders 6x
 Precast maxi: 8 cylinders 6x or 6 cylinders 9x

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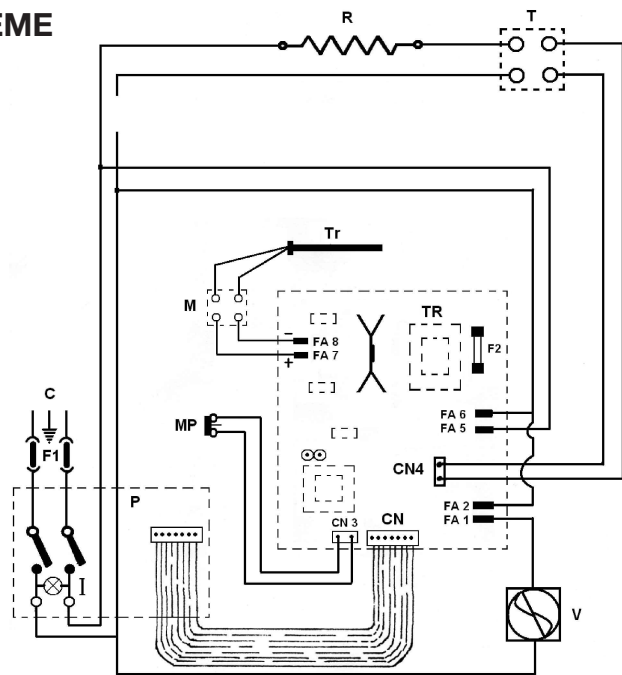
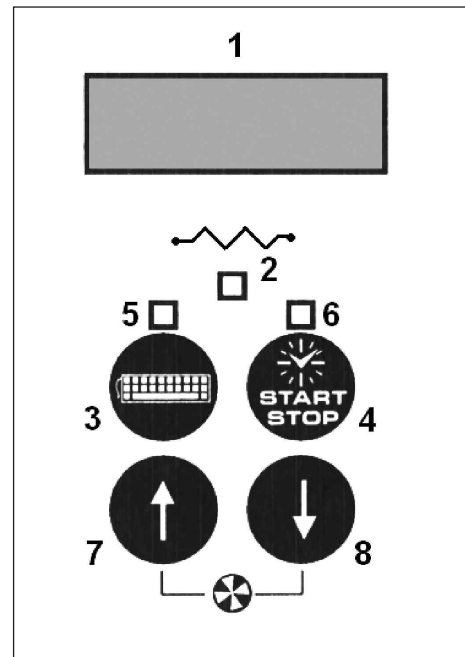
PANNELLO COMANDI / CONTROL PANEL

- 1 DATA DISPLAY
- 2 ORANGE LED / ACTIVATED RESISTANCE
- 3 KEY FOR CYCLE PROGRAMMING
- 4 KEY FOR START UP PROGRAMMING / KEY FOR INTERRUPTING THE CYCLE
- 5 RED LED / PROGRAMMING GOING ON
- 6.a RED LED / DISPLAYING THE POSTPONED TURN-ON
- 6.b ORANGE LED / DISPLAYING THE RISE IN TEMPERATURE
- 6.c GREEN LED / DISPLAYING HOLDING TIME
- 7 KEY FOR VALUE INCREASING
- 8 KEY FOR VALUE DECREASING

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CONGRATULATIONS FOR YOUR CHOICE!!

Choosing this apparatus you have purchased a particularly reliable unit that will facilitate your work.

IMPORTANT !

Before installing the apparatus read carefully this manual and keep it for future consultations.

WARNING

Do follow carefully all the indications brought back in this manual. During all the maintenance operations disconnect the power cord from the electrical outlet, and/or disconnect the compressed air and/or the water attack and/or other attacks of external parts to the apparatus. Do not introduce stranger objects inside the unit during the maintenance operations. Use only the accessories in equipment for the filling operations and/or maintenance. Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not contemplated in this manual, can involve risks. For operations of this type, contact the technical service. Before turning on the unit and connecting the power cord to the electrical outlet, make sure that the voltage corresponds to the one brought back on the fabrication nameplate placed on the back part of the apparatus.

CAUTION:

- **Do not make any maintenance operation when the power cord is connected: FULGURATION DANGER!**
- **Use only genuine spare parts. Placing not genuine spare parts can void your warranty, moreover can make the apparatus not in compliance with the norms, and dangerous when in use.**
- **For whichever inherent information concerning installation, maintenance and use of the unit, contact the customers' service TISSIDental:**

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Fax (+39) 02 98285327 - Tel. (+39) 02 98282694 - www.tissidental.com

UNPACK

Verify integrity of the pack at the moment of the reception of the apparatus. Verify that in the packaging all the parts and the supplied accessories of series are contained together with the apparatus.

1 SCOPE OF MANUAL

This manual has been written by the manufacturer and it is an integral part of the machine's equipment. Before installing or using this machine, the instruction found on this manual must be read carefully.

This manual must be kept for future reference.

2 GENERAL CHARACTERISTICS / EXPECTED USE

The oven allow to program a cycle composed by a timer phase and three working phases according to the operator's needs thanks to free data input within the programming ranges.

Data Chart

	Timer
Programmable Data	Programming Range
Clock	From immediate start to 99 hrs. and 59 min.
Phases 1,2 (Heating)	
Programmable Data	Programming Range
Temperature	From 0°C to 1100°C
Increase gradient	From 0°C/min. to 30°C/min.
Holding time	From 00.00 (hrs.min.) to 20.59 (hrs.min)

Phase 3 (Heating)	
Programmable Data	Programming Range
Temperature	From 0°C to 1100°C
Increase gradient	From 0°C/min to 30°C/min
Fan turn-off temperature	From 1°C to 1100°C
Or	
Fan turn-on time	From 1 minute to 120 minutes

In the PRECAST oven series it is also possible to program a cooling phase. In this specific case the cooling speed is not carried out by the oven.

Expected use: *Equipment for exsiccation and baking of refractory coatings of the cylinders for the fusion of the metal prosthetic elements.*

3 INSTALLATION

- Verify the apparatus has not suffered any evident damage during transportation.
- Place the apparatus on a stable horizontal non flammable surface away from any water sprayings and from flammable materials. The oven must be placed under an extracting hood that allows outside emission of fumes produced during preheating respecting the environment and current laws.
- Before installation make sure the supply mains are in conformity with tag (230 Vac – 50/60 Hz) and that the power line is protected with a differential thermo magnet switch. Also check electrical inputs and power installed. If you are not sure what kind of power is available to you please contact the Electric Power Company. In particular, verify if the grounding system is efficient.
- Make sure the general switch is in the off position (0), then connect the electric cable’s plug to an electrical outlet.

4 PROGRAMMING OF CYCLE

To program a cycle follow correctly and in the indicated order the instructions below:

- 4.1 Turn-on machine
- 4.2 Program data for phases 1,2 and 3
- 4.3 Program fan’s turn-off temperature or fan’s turn-on time.

The programming of fan’s turn-off temperature or the fan’s turn-on time can be done only if the oven has a fan extracting chimney.

4.1 Turn-on machine

The oven is turned-on by turning the general switch to the On position (I).
 The interior green light will be lit.
 The word OFF will appear on display. All leds will be off.
 In case numbers would appear on display, press START/STOP button to put oven on state of IDLE-OFF.

4.2 Program data for phases 1,2 and 3

For phases 1 and 2 the user can program in sequence: temperature, increase gradient and holding time, while for phase 3 the user can program temperature and increase gradient.

The choice of phase to be programmed is up to the user’s discretion: three phases may be programmed or some phases can be by-passed. **The operator must however always set phase 3 because in this phase the final temperature is kept until the cycle’s stop using the START/STOP button.**

Example:

- To set 3 phases, program phases 1,2 and 3.
- To set 2 phases, program phases 2 and 3.
- To set 1 phase, program phase 3.

To enter the program press key . The key's  red led will lit intermittently for all time of programming. On display the words “Ph.1”, “°C” and the last temperature value of phase 1 programmed will appear in sequence.

The change of this parameter (programming range 0/1100°C) is carried out using key  (to increase value) and  (to decrease value).

In case a value equal to 0 is set this phase is by-passed (no parameters are considered for this phase) and cycle starts directly from phase 2.

Press key  to confirm entry.

On display the symbol “°C-M” and the last speed increase value programmed will appear in sequence.

The modification of this parameter (programming range 0/30°C/min) is carried out using keys  (to increase value) and  (to decrease value).

Setting this value at 0, will increase temperature to its maximum speed (resistance always active).

Press  key to confirm entry.

On display the word “Hr.Mn” followed by the last holding time programmed will appear.

Four numbers divided by a dot will appear on display.

The two still numbers on the left side of the dot indicate the hours, while the flashing numbers on the right side of the dot indicate the minutes.

The first parameter to change are the hours.

The modification of this parameter (programming range 00/20 hours) is carried out by using key  (to increase value) and  (to decrease value).

Press  key to confirm entry and pass to programming of minutes.

On display you will see now the two numbers on the right side of the dot (indicating minutes) still while the two numbers on the left side of the dot (indicating the hours) flashing.

The modification of this parameter (range of programming 00/59 minutes) is carried out using keys  (to increase value) and  (to decrease value).

In the case in which a value equal to 00.00 is set, the time of preheating is not included and the oven after reaching the temperature will pass directly to phase 2.

Press key  to confirm data entry and pass to programming of phase 2.

The above described procedure is repeated for phase 2, while for phase 3 the parameters to be set are temperature and increase gradient.

After programming the increase gradient for phase 3, go to programming of fan.

4.3 Programming fan turn-off temperature or fan turn-on time

The operator has two options to program fan:

- set fan turn-off temperature

or

- set fan turn-on time

Set fan turn-off temperature

In this specific case the fan starts when the oven goes into working phase (heating) and it turns-off at the temperature set.

The programming is done after confirming the setting of increase gradient of phase 3 using the  key.

On the display you will now see in sequence the words “SUCt” and “toFF” and after some seconds the last turn-off temperature programmed is shown.

The modification of this parameter (programming range 0/1100°C) is carried out using key  (to increase value) and  (to decrease value).

Pressing key  will confirm the entry and will quit programming. You will see OFF on the display.

Set fan turn-on time

In this case the fan is activated during working phases (heating) at any time by pressing simultaneously keys

↑ and ↓.

The programming of this data can be done only if fan turn-off temperature has been set at 0 (toFF=0) and have confirmed the entry using the  key.

On the display will appear the words “SUCt”, “tIME” in sequence and after some seconds the last fan turn-on time programmed will be shown.

The modification of this parameter (programming range 1/120 minutes) is carried out using key ↑ (to increase value) and ↓ (to decrease value).

Pressing key  will confirm entry and will quit programming. You will see OFF on the display.

The starting of the fan can be activated ONLY during working phases (heating) by pressing simultaneously keys ↑ and ↓ and it will stay in function for the duration of programmed time.

In case the operator wants to interrupt the functioning of fan before the set time is over, keys ↑ and ↓ must be pressed simultaneously.

Of course programming this data is only logical if the oven has an extracting chimney.

The PRECAST oven series are readied for application of chimney with extracting fan.

The connection of the oven to the extractor must be done by competent and authorized personnel, with the general switch in the off position (0) and the electrical cable disconnected.

In the back of the oven there is an extractor connection outlet.

The data set will remain in memory.

The operator can however, change the set data at any time desired by entering the program.

5 START UP PROGRAMMING

After programming the cycle the operator is given two options:

- immediate turn-on
- postponed turn-on

During programming of start-up the  key's red led will start flashing intermittently.

Immediate turn-on

With oven in OFF press the START/STOP key.

Four zeroes divided by a dot will appear on display.

Press twice (2 times) the START/STOP key and the cycle is activated with an immediate effect.

Posponed turn-on

With oven in OFF press the START/STOP key.

Four zeroes divided by a dot will appear on display.

The two still zeroes on the left side of the dot indicate the hours; while the two flashing zeroes on the right side of the dot indicate the minutes.

The first parameter to change are the hours.

The modification of this parameter (programming range 00/99 hours) is carried out using keys ↑ (to increase value) and ↓ (to decrease value).

Press the START/STOP key to confirm entry and to go to programming of minutes.

On display you will now see two still numbers to the right side of the dot (indicating the minutes) while the two numbers on the left side of the dot (indicating the hours) are flashing.

The modification of this parameter (programming range 00/59 minutes) is carried out by pressing the keys ↑ (to increase value) and ↓ (to decrease value).

Press key START/STOP to confirm entry.

In this specific case, after setting the hours and the minutes, on the display will appear the time in hours and minutes corresponding to the residue time to the start of the cycle. Such value is shown as countdown to start-up time.

For all duration of timer, the small dot between the four numbers will flash and the red led of START/STOP key will stay on.

6 BAKING CYCLE

This cycle begins after programming the start-up (see chapter 5), thus, after pressing three consecutive times the START/STOP key (immediate turn-on) or after the programmed time is over (postponed turn-on).

At moment of cycle's start-up, a beep will let the operator know the passage from timer phase to working phase. On the display it will be shown for a few seconds the programmed temperature followed by the real temperature found in the chamber.

The START/STOP key's orange led will stay on until the set temperature is reached, the  orange led activated resistance will lit indicating the power delivery to resistance.

At the moment the set temperature is reached, a beep will notify the operator and on the display just for a few seconds the time holding value set for the phase in course followed by the real temperature on the oven's chamber will be shown.

The green led of the START/STOP key will remain on for the duration of the time set; the orange  led of activated resistance stays on indicating the power delivery to resistance for keeping up temperature.

At each passage from one phase to another, a beep will notify the operator.

At the time the set temperature is reached for phase 3, a longer beep will notify the operator the end of the cycle. The temperature inside the baking chamber is kept until the START/STOP key is pressed to turn the oven OFF.

At any time of the cycle the operator can interrupt the ongoing cycle by pressing the START/STOP key and turning the oven OFF.

7 INSPECTIONS

IDLE OFF state

In the IDLE-OFF state the following inspections may be done, using the indicated key:

-  : by keeping this key pressed you may see on the display the temperature inside the baking chamber.

Phases 1 and 2

During phases 1 and 2 the following inspections are possible by pressing the indicated key:

-  : by keeping this key pressed you may see on the display the temperature set for the phase in course.
-  : by keeping this key pressed you may see on the display the speed of increase set for the phase in course.
-  : during the increase of temperature programmed, by keeping this key pressed, the holding time programmed is shown. During the holding time by pressing this key the residue holding time for the phase in course is shown.

Phase 3

During phase 3 the following inspections are possible by using the indicated keys:

-  : by keeping this key pressed the set temperature for the phase in course is shown.
-  : by keeping this key pressed the increase speed set for the phase in course is shown.

In all these cases at the release of the key, after a few seconds the value present before pressing the key will appear.

8 POWER FAILURES

In case of power failures, the following situations present:

During TIMER phase

When power returns, you will see on the display for a few seconds the residue time to the start-up of oven memorized at the time of the failure, afterwards, the countdown starts again (but the time the power missed due to the failure is not considered).

The operator may decide to:

- let the timer finish its course
- put timer on zero by pressing the START/STOP key, turning the oven OFF and activating the cycle with immediate effect by pressing three consecutive times the START/STOP key.

During increase on phases

When power returns, the cycle continues from the phase in which it was at the moment the power failed and it will recuperate the lost temperature.

During stationary time on phases

When power returns two cases may present:

- If the time when power was missing was minimal the temperature inside the chamber did not decrease, and the oven will do all over again the holding time programmed.
- If the temperature in the chamber decreased, the oven will reach the temperature programmed and all over again the holding time programmed.

In each of these cases when the power returns a beep will notify the operator of the problem caused.

During programming

When power returns the oven will be at IDLE OFF showing on display the word OFF.

9 ALARMS**9.1 “oPEn” open door Alarm**

At any phase of the baking cycle, if the operator opens the oven’s chamber door the power to the electric resistance will be disactivated.

On the display you will see the word “oPEn” flashing for all the time the door stays open.

Normal functioning is restored after closing the door.

9.2 “Over” Temperature Alarm

- The temperature alarm is activated each and every time temperature of 1150°C is exceeded for a period of time equal or longer than 30 consecutive seconds.
- The temperature alarm is activated in case of non-connected thermocouple (detected after 30 seconds).

On the display you will see the word “oVEr” flashing.

A buzzer will notify the operator this problem and the resistances will be disactivated.

In this case the alarm will be turned off if the oven is turned off with the general switch in the off position (0).

Naturally the cause of the alarm is not resolved, the alarm will sound again when the oven is turned- on.

Contact our Technical Department in order to solve this problem.

9.3 “thCO” Inverted Thermo couple Alarm

In the case in which the thermo couple were installed with inverted wires, the word “thCO” will be shown flashing on the display.

A beeper with intermittent sound will notify the operator this problem and the resistances will be disactivated.

To reset the alarm, the wires in the thermo couple must be inverted. This operation must be done by competent and authorized personnel with the general switch in the off position (0) and with the electrical cable disconnected.

10 ENVIRONMENTAL TECHNICAL DATA

Use: Indoors

Temperature: from +5°C to +40°C

Relative Humidity: 80% maximum

Altitude: up to 2000 meters

Installation category: II

Degree of pollution: 2

11 ORDINARY MAINTENANCE

The PRECAST oven series do not need any particular maintenance.

Only in case of prolonged inactivity the buffer battery found on the board may get discharged thus it is necessary to be substituted (see chapter 9.4). It is therefore, advisable, to make the oven work periodically.

12 WARNINGS

- For a proper and lasting functioning of this oven it is recommended not to use it for long periods at the maximum temperature.
- It is recommended, before turning off the oven with the general switch in the off position (0), to press the START/STOP key to put oven in IDLE-Off state which is shown with the word "OFF" on the DATA display.
- During the use of oven the operator must wear gloves and safety accessories, as the oven reaches very high temperatures both inside and outside.
- It is indispensable to use the right pliers and gloves to take out and put cylinders inside baking chamber.
- For cleaning operations use a dry cloth and a liquid detergent (do not use flammable liquids). Dry with a soft cloth. Do not place for any reason the oven under a spout of water. Cleaning operations must be done with the general switch off (0) position, with the electrical cable disconnected and the oven cold. Before adopting any cleaning method, except for those recommended by the manufacturer, the user must contact the manufacturer to verify if the proposed method does not pose risk of damage to the apparatus.
- Do not place any objects over the electrical cable and do not put cable in places where it can be stepped over.
- In case of prolonged inactivity disconnect the electrical cable.
- For any needs of transportation, always use the original packaging and do not transport oven when it is still hot.

13 IMPORTANT INFORMATION

- The Manufacturing Company declines any responsibility for any apparatus installed, utilized, modified, repaired and/or altered by non competent and unauthorized personnel.
- The apparatus has been designed and built only for the uses contemplated, consequently the Manufacturing Company refuses any responsibility for damages to persons, animals, things, and the machine itself caused by improper use of this apparatus.
- The improper use of this machine is dangerous!!
- Subject the apparatus to a complete annual revision by the Manufacturer.
- To clarify any doubt and/or technical explanation please contact our Technical Department.

14 GUIDE TO COMMON QUESTIONS

The apparatus is not turning on:

- Control that the power cord plug is properly connected to the electrical outlet.
- Check that power is on at the electrical outlet.

Carried out these controls, in case the apparatus still presents disadvantages, contact the customers' technical service.

15 SPECIFICATIONS - DIMENSIONS - WEIGHT

PRECAST MAXI

2.850 Watt – 6 x 32 mm F 16 A/250V – Kg 35
MIS EXT 39X52X50 – MIS INT 16.5X26X11

PRECAST MED

2.400 Watt – 6 x 32 mm F 16 A/250V – Kg 30
MIS EXT 33X36X49 – MIS INT 16X16X11

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1.700 Watt – 6x 32 mm F 8 A/250V – Kg 20
MIS EXT 25X32X45 – MIS INT 12X16X9

16 NOTES

- For the external cleaning of the device use a cotton tawel damped with water.

- The device can work in laboratory room from 0°C to 35°C.

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